

ProCare Services

Our ProCare Services solutions help optimize the reliability and life of your equipment. Our service plans are designed to make it easier for you to get the most out of your investment in equipment from Stryker with options ranging from unlimited priority repairs to preventive maintenance. Please contact your Surgical sales representative for detailed ProCare Services options.



Optimized power tool management

Originated through research and guided by our customers, we are pleased to offer Smart Equipment Management (SEM), a revolutionary, web-based application designed to help you optimize the management of your Stryker F1 System through device-derived, actionable data.

- Proactive device maintenance
- Balanced product use
- Spend justification
- Simplified service
- Asset tracking



stryker

Stryker F1 System part numbers

SmartGRIP Modules

1900-013-000	SmartGRIP Pencil
1900-012-000	SmartGRIP Pistol

Motors

1900-034-000	Sagittal saw
1900-031-000	Oscillating saw
1900-037-000	Reciprocating saw
1900-015-000	Micro drill
1900-099-000	Universal motor

Attachments

1900-062-000	Wire collet
1900-110-000	A/O chuck
1900-132-000	5/32" keyed chuck
4100-130-060	Universal Jacob's key

Battery charger

7110-190-000	Stryker F1 Universal Battery Charger
1900-125-000	Charger module

Sterilization solutions

1900-476-000	Perforated container S bundle
1900-376-000	Case bundle
1900-576-000	Perforated container bundle
1900-676-000	Solid container bundle
1900-476-020	Bottom container S - 1/2 size x 6"H
1900-476-030	1/2 size container S lid
1900-376-010	Handpiece insert tray
1900-377-010	SmartGRIP insert tray
7102-453-040	Case bottom and lid
7102-553-030	Perforated container lid
7102-553-020	Perforated container bottom
7102-653-020	Solid container bottom

For questions or to place your order today, please contact your Surgical sales representative or call 800 253 3210.

The information presented is intended to demonstrate the breadth of Stryker's product offerings. A healthcare professional must always refer to the package insert, product label and/or instructions for use before using any of Stryker's products. Products may not be available in all markets because product availability is subject to the regulatory and/or medical practices in individual markets. Please contact your Stryker representative if you have questions about the availability of Stryker's products in your area.

See Stryker F1 Cordless Small Bone Power Tool System Instructions for Use (IFU) for complete details.

Stryker Corporation or its divisions or other corporate affiliated entities own, use or have applied for the following trademarks or service marks: ProCare, SmartGRIP, Stryker, Stryker F1. All other trademarks are trademarks of their respective owners or holders.

© 2017 Stryker | www.stryker.com

Stryker Instruments
4100 East Milham Avenue
Kalamazoo, MI 49001 USA
t: 269 323 7700 f: 800 999 3811
toll free: 800 253 3210

Literature number: 9100-003-555 Rev B PF/PS

stryker

Free to be your best

Stryker F1

Cordless Small Bone Power Tool System



Stryker F1® Cordless Small Bone Power Tool System

The Stryker F1 System is a leap forward in **balanced, lightweight power and precision** – so you're free to be your best.

SmartGRIP Technology



Pencil grip



Pistol grip

The electronics of a cord, console and handpiece are combined in a single device to produce Stryker's exclusive SmartGRIP Technology.

- Exceptional maneuverability and control
- Minimized chance of sterility breaks
- Proven durability with extensive trials to ensure reliability and integrity in a wide variety of potential scenarios*



Motors

The Stryker F1 System offers five motors, all interchangeable between SmartGRIP Module gripping styles. This versatility allows for customizable sets, so you have only the components you need.



Sagittal saw
1900-034-000



Oscillating saw
1900-031-000



Reciprocating saw
1900-037-000



Micro drill
1900-015-000



Universal motor
1900-099-000



Wire collet
1900-062-000



A/O chuck
1900-110-000



5/32" keyed chuck
1900-132-000



Universal Jacob's key
4100-130-060

Attachments

The Stryker F1 System offers four attachments for use with the universal motor, allowing for more efficient operation with fewer components.

SmartGRIP Reliability

- SmartGRIP Modules can remain on the shelf, unused, for 30 days without losing power and performance*
- Insulation within each battery increases resistance to thermal damage during exposure to high temperatures throughout sterilization



Pencil grip



Pistol grip

Battery indicator light

- The module-ready light uses colors to display battery status information:
 - Solid green light appears upon insertion of a motor to indicate that the battery is charged, the motor is connected, and the SmartGRIP Module is ready for operation
 - Solid amber light appears to indicate that the battery charge is low
 - Blinking amber light appears to indicate that the battery is expended and the SmartGRIP Module is about to shut down




Consistent power and uptime

- High-capacity cells deliver improved performance to get you through the toughest case*
- Multiple osteotomies performed without having to re-charge the SmartGRIP Modules*



Sterilization options

- The Stryker F1 System gives you an all-inclusive container with two insert trays to hold every component in the system
- SmartGRIP Modules may be sterilized in multiple ways, including an approved autoclave cycle**

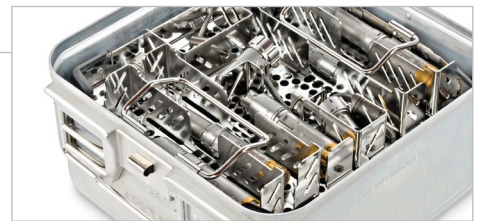

Cutting accessories

- Extensive range of burs, blades and rasps are available for use with the Stryker F1 System
- Exclusive offering: micro dual-cut blades and resurfacing tool burs for cutting and shaping bone



*Stryker engineering data on file. Kalamazoo, MI.

**For complete cleaning and sterilization instructions, please refer to Stryker F1 Care Instructions.

Signature History Report

Version	Signature	Role	Event Date	Vote
B	Atkins, Dilara	CA_Owner	2017-10-19 12:07:20 EDT	Continue
B	Atkins, Dilara	CA_Marketing Communications	2017-10-19 14:02:00 EDT	Approve
B	VanderGenugten, Rachel	CA_Marketing Manager	2017-10-19 15:11:53 EDT	Approve
B	Khurmi, Paminder	CA_Regulatory Affairs	2017-10-23 09:13:58 EDT	Approve
B	Brewer Lindsey	CA_Legal	2017-10-23 11:54:14 EDT	Approve
B	Karl, Jeffrey	CA_Product Design Engineer	2017-10-25 08:42:27 EDT	Approve

NOTE: IF THIS REPORT IS BLANK, PLEASE SEE WINDCHILL ECN FOR SIGNATURES.

Related Information

CR Number	CR0334018
CR Class	Collateral Material Only
Originating Entity	INST: Kalamazoo
Origin	
Initial CR Approval Date	10/17/2017 16:45:11
CN Number	CN181535
CA Number	CA220187
NPD Disposition Plan	n/a